Ice-lined Refrigerator Model MK 304

GB Instructions for use
FR Consignes d'utilisation
ES Instrucciones para el uso
RUS Инструкция по эксплуатации
CHI 使用说明

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General Information

The MK-Model is an ice-lined vaccine refrigerator. It has one compartment for vaccine storage (+2 to +8°C) equipped with baskets.

Power supply

The MK 304 is equipped with a heavy-duty compressor that can operate at voltages 22% below nominal voltage. See the rating plate. If voltage variations go beyond this range, it is recommended to use an automatic voltage stabilizer.

Technical data

See the rating plate, which is placed at the rear side of the appliance.

Unpacking and installation

Unpacking

1. Unpack the appliance and check that it has not been damaged. If you observe any damage, inform your supervisor.
2. Check the rating for correct voltage and frequency. (fig. 7 page 10)
3. Open and remove all internal packing material.

Contents

General information	2
Power supply	2
Technical data.....	2
Unpacking.....	2
Get to know your refrigerator	3
Installation	4
Electrical connection.....	5
Operation and function	6
Cool down of the appliance	6
Loading the appliance	7
Maintenance and cleaning.....	8
Trouble shooting	9
Warranty, Spare parts and service	10
Wiring diagram	13
Disposal	14

Reserving the right to alter specification without prior notice.

Get to know your Refrigerator

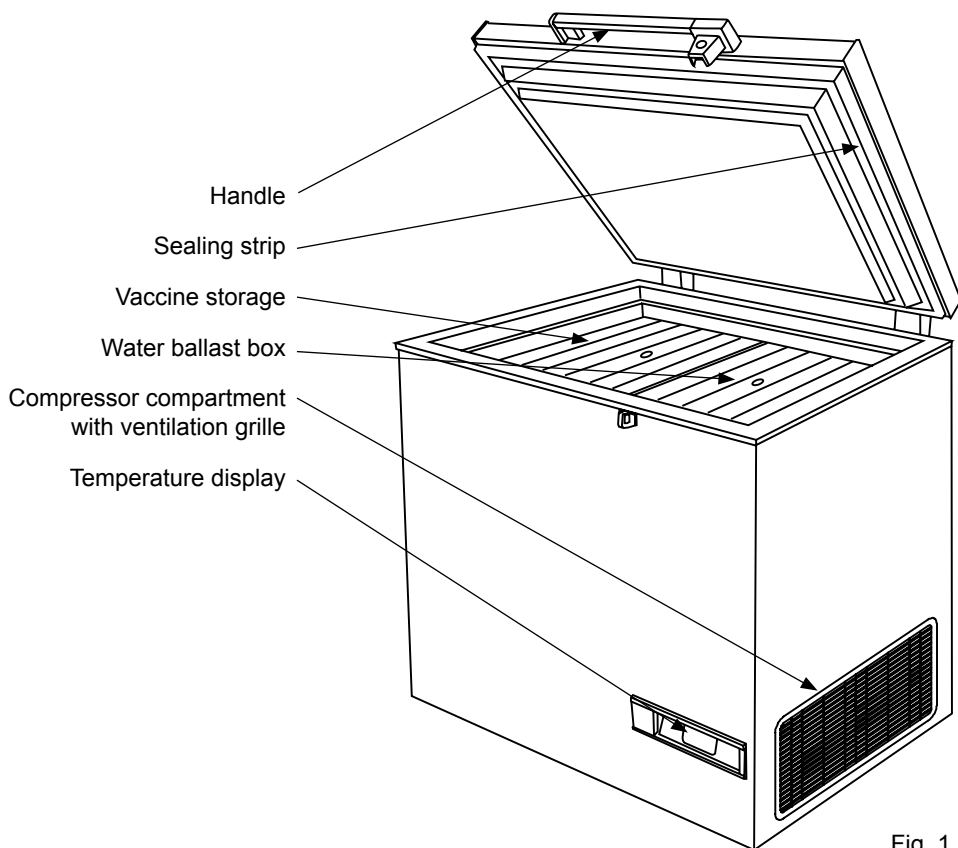
Fig. 1

Installation

Location

The appliance must be placed in a well ventilated room, not in direct sunlight and away from other heating sources.

When installing the appliance please ensure that the floor is level. Please see figure 2 for further directions.

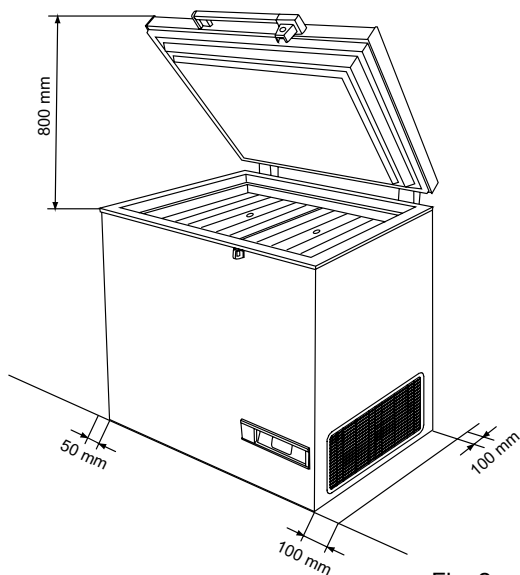
Fig. 2

There must be at least 30 mm clearance between the base of the appliance and the floor.

IMPORTANT! There must be free access to the ventilation grille (fig. 3).

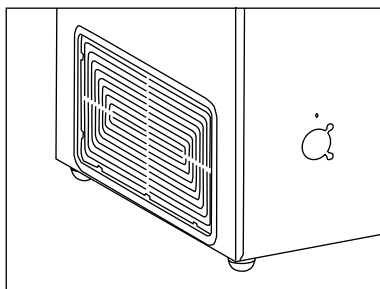
Fig. 3

Electrical connection

Preparation

Check that the plug fits your type of socket. If it does not fit, have a qualified electrician to fit the plug. This appliance must be earthed.

Colour code for leads:

Lead of cord		Pin of plug
230 V	115 V	
Brown	White	Live
Blue	Black	Neutral
Green/Yellow	Green	Earth

Connect the plug to the socket and switch on the appliance.

- Check, by listening, that the compressor is running, if it does not run, contact your supervisor.

Operation and function

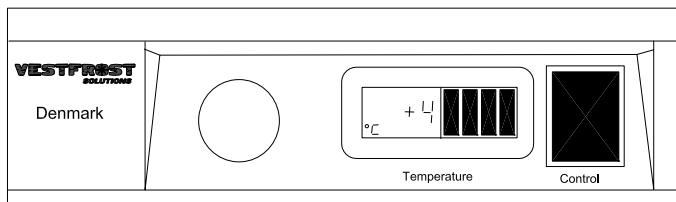
Fig. 4

Electronic thermostat

The appliance has an electronic thermostat that controls the temperature in the vaccine compartment. The thermostat is set by the factory

1. The green lamp is on when the appliance is connected to the power supply.
2. Thermometer.

Starting procedure

Connect the mains lead to the power supply. The green lamp must light (fig. 4) to indicate that the appliance is operating. The electronic thermostat makes a self test, 20 seconds before the compressor starts.

Temperature control

It is recommended to check the inside temperature with an accurate thermometer once a day. The inside temperature must be checked regularly according to WHO's standards and specifications.

Cool down of the appliance

Before the appliance is loaded with vaccines the ice-lining in the 4 sides must be frozen. To ensure that the ice-lining in the 4 sides is frozen do the following:

1. Place a thermometer in the top basket.
2. Plug in the appliance, and let it run for 24 h.
3. Check the temperature in the top and bottom basket (must be between +2° and +8°C).

The temperature in the vaccine compartment must always be monitored on the thermometer and be within the range +2° to +8°C. Due to tolerance of the thermostat you should always control the temperature during cool down.

Loading the appliance

Loading vaccines

When the temperature in the vaccine compartment has stabilized, i.e. and the temperature is between $+2^{\circ}$ and $+8^{\circ}\text{C}$ and the compressor stops and starts, vaccines can be loaded. The vaccines should be placed and arranged in the baskets, and remember to place the water filled ballast lids at the top of the upper baskets. (fig. 5)

To ensure air-circulation and prevent too low vaccine temperature, direct contact to the inside walls must be avoided. The Vaccine load in the bottom basket shall not be higher than 250 mm.

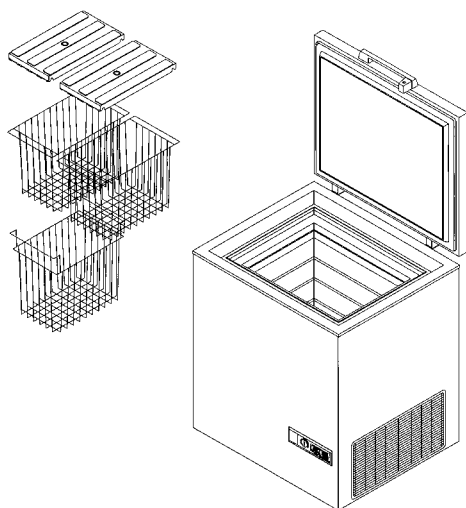
Fig. 5

Do not load lower baskets
above this height (250mm)

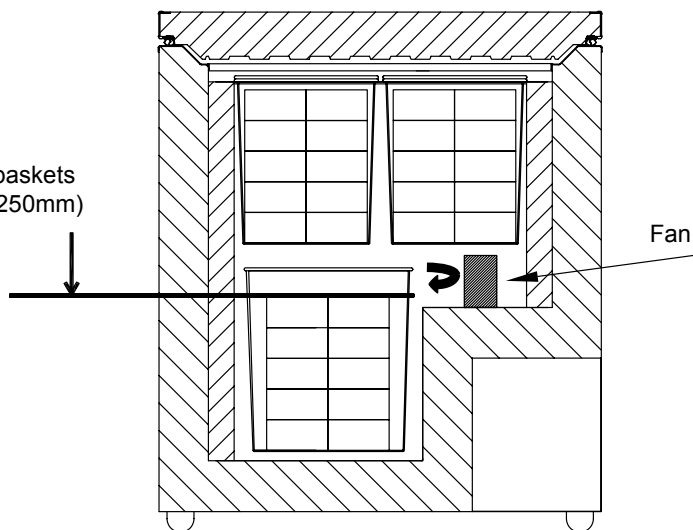
Fig. 6

Maintenance and cleaning

Daily maintenance:

The temperature in the vaccine compartment must always be monitored on the thermometer every day. Please note that the vaccine in the appliance may not freeze.

Weekly maintenance:

During normal use, water can accumulate at the bottom of the appliance; remove with a cloth or through the drainage hole.

Wipe of water droplets on the inside wall at the same time.

Under certain ambient conditions, rime can form on the inside wall. Remove the rime without using sharp edged tools.

Check if lid gasket is sealing tight to the top frame when the lid is closed. A tight sealing lid reduces accumulation of water and formation of rime significantly.

Monthly maintenance:

Clean the grill on the right side of the refrigerator once each month.

Yearly maintenance:

Electric connections and components are to be checked and cleaned once a year or more if necessary.

Cleaning

Disconnect the power supply before cleaning. The best way to clean the appliance is by using luke warm water with a small amount of unscented detergent. Never use cleaning agents that scour. Use a soft cloth. Rinse with clean water and dry thoroughly. It is important to prevent water from running into the control panel.

The sealing strip around the lid must be cleaned regularly to prevent discolouration and prolong service life. Use clean water. After cleaning the sealing strip, check that it continues to provide a tight seal.

If the appliance is not being used for any period of time, switch off the appliance, disconnect the power supply, empty the appliance, clean the inside, and leave the lid open to allow air circulation and prevent smells.

Trouble shooting

Fault	Possible cause	Remedy
Compressor is not running, and the ice packs are not cold	Be patient, it is most likely that the compressor will start within a few minutes.	If this is not the case, check the following: - Check that power is connected. - Check the fuse and replace it if necessary. - If the above is OK, call technical supervisor.
Compressor is running, and the temperature is too high	The ventilation grille is blocked. The lid is not closed properly. The temperature in the room in which the appliance is installed is too high.	Ensure unhindered air circulation. Ensure that the lid is closed properly. Shield the appliance against direct sun light and ensure more ventilation to the room.
No temperature is displayed	The thermometer is broken. There is not enough light for the solar sensor.	Change the thermometer. Turn on the light.
Temperature goes below +2°C in a specific basket	Air circulation is restricted. Basket in direct contact with wall.	Check load height is not above 250mm (or loadline) in lower baskets. Move the end basket a little to leave a gap between the basket and the wall.

Warranty, spare parts and service

Warranty disclaimer

Faults and damage caused directly or indirectly by incorrect operation, misuse, insufficient maintenance, incorrect building, installation or mains connection. Fire, accident, lightening, voltage variation or other electrical interference, including defective fuses or faults in mains installations are not covered by the warranty.

Repairs performed by others than approved service centres and any other faults and damage that the manufacturer can substantiate are caused by reasons other than manufacturing or material faults are not covered by the warranty.

Please note that changes to the construction of the appliance or changes to the component equipment of the appliance will invalidate warranty and product liability, and the appliance cannot be used lawfully. The approval stated on rating plate will also be invalidated.

Spare parts

When ordering spare parts, please state the type, serial and product numbers of your appliance. This information is given on the rating plate. The rating plate contains various technical information, including type and serial numbers.



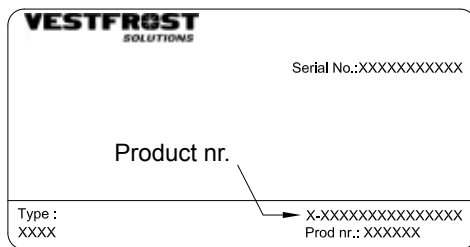
Fig. 7

MK 304

Position	Item number	Item name
0061	0-6538001	Filter drier,
0070	A921001	Cover + wirring clamp
0087	0-6038175	Base plate fittings, set, complete
0129	8-036038480	Compressor - FR 8,5 G,HST Complete, R134a
2168	0-703930206	Display with remote control - EPT-R1
5000	3010049	Drain plug
5002	6000959	Foot with pin
5118	5000904671	Lid foamed without handle/hinges
5119	3020107-01	Rubber gasket
5372	A93010210	Water ballast box
5401	1510031	Hinge 30
5450	3010040-01	Top part for hinge cover
5451	3010032-01	Bottom part for hinge cover
5605	304090501	Handle with lock (push and turn)
5606	8090342-94	Inlay for handle,
5682	1510046	Key, set - 2pc.
5683	2040145	Catch for handle
5684	3010265-01	Cover for catch
5685	600098801	Lock with keys,(push and turn)
5714	7020144	Thermostat phial (Fan box)
5714	7510582	Thermostat phial
5717	7020293	Thermostat
5738	7010027	Pilot lamp, green
5756	7020164	Thermometer, solar
5783	3040141	Control panel
5812	6520106	Starting device, 117U6015
5850	6520004	Starting condenser 117U5015
5870	6000902	Fan
5876	7090047	Fan box, complete, inclusive heating element
5907	3510024	Basket
5927	3510023	Basket
5963	A93010357	Eutectic element
5970	3010308-01	Motor screen
6553	7535008	Mains lead
6760	7030108	Transformer, 230/2X12 DC
7100	7030333	Fan

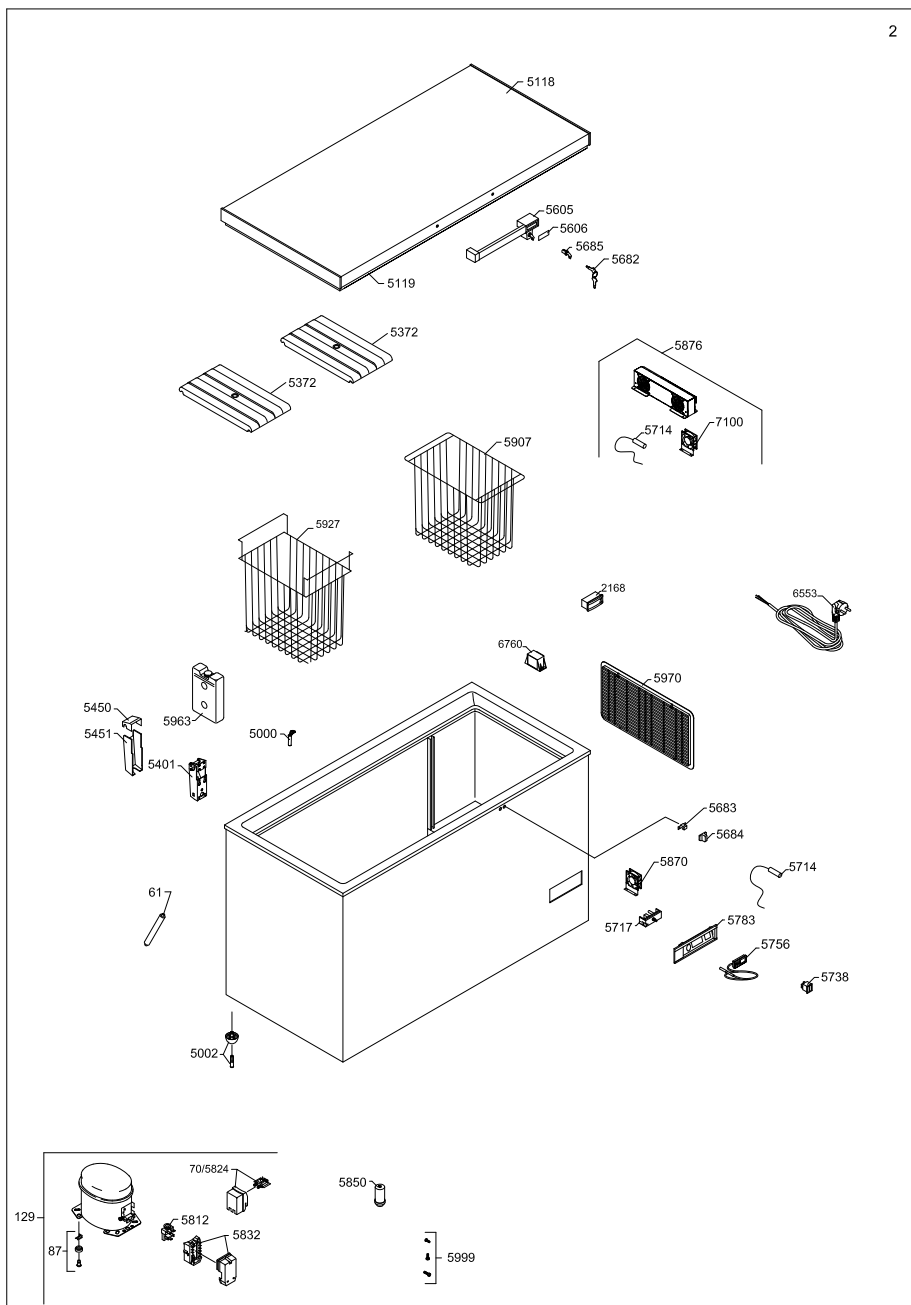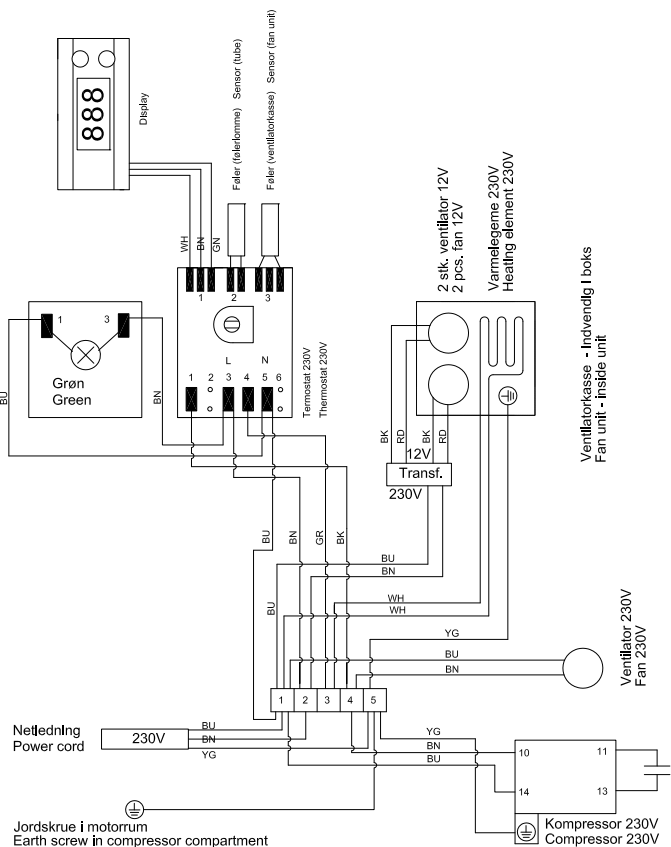Please Note! The drawing covers all variants of the model and therefore contains more pos. numbers than your product. See the spare parts list for exact product specification.

	Farbe	Farbkode	Colour Code	Farb kode
-				
BN	Braun	Brown	Brown	Braun
BU	Blau	Blue	Blue	Blau
BK	Schwarz	Black	Black	Schwarz
LI	Lila	Black	Black	Lila
YG	Grün	Yellow / green	Yellow / green	Grün
WH	Gelb	White	White	Gelb / grün
WH	Gelb	White	White	Gelb
WD	Rot	Red	Red	Rot
GN	Grün	Green	Green	Grün
RD	Rot	Red	Red	Rot
GN	Grün	Green	Green	Grün

Start kondensator
Starting capacitor

Wire-diagram

1006261-01

Disposal

Information for Users on Collection and Disposal of Old Equipment and used Batteries

These symbols on the products, packaging, and/or accompanying documents mean that used electrical and electronic products and batteries should not be mixed with general household waste. For proper treatment, recovery and recycling of old products and used batteries, please take them to applicable collection points, in accordance with your national legislation and the Directives 2002/96/EC and 2006/66/EC.

By disposing of these products and batteries correctly, you will help to save valuable resources and prevent any potential negative effects on human health and the environment which could otherwise arise from inappropriate waste handling.

For more information about collection and recycling of old products and batteries, please contact your local municipality, your waste disposal service or the point of sale where you purchased the items.

Penalties may be applicable for incorrect disposal of this waste, in accordance with national legislation.

For business users in the European Union.

If you wish to discard electrical and electronic equipment, please contact your dealer or supplier for further information.

[Information on Disposal in other Countries outside the European Union]

These symbols are only valid in the European Union. If you wish to discard this product, please contact your local authorities or dealer and ask for the correct method of disposal.

Note for the battery symbol (bottom two symbol examples):

This symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

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